## 24H1 MyCRO Band

|  |    |
|--|----|
| <b>DE</b> Gebrauchsanweisung (Benutzer) .....            | 2  |
| <b>EN</b> Instructions for use (user) .....              | 9  |
| <b>FR</b> Instructions d'utilisation (Utilisateur) ..... | 16 |
| <b>NL</b> Gebruiksaanwijzing (Gebruiker) .....           | 24 |
| <b>CS</b> Návod k použití (Uživatel) .....               | 31 |

## INFORMATION

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### INFORMATION

Date of last update: 2022-02-25

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the qualified personnel.
- ▶ Please keep this document for your records.

The Ottobock MyCRO Band is an orthosis to aid in the correction of head shape caused by positioning. The orthosis uses contact and growth zones to guide the growth of the head. The contact zones define gentle limits for growth, while the growth zones leave space in areas required for forming the natural head shape. An adaptable closure allows for adjustability as the child grows. The orthosis is made of thermoplastic material with a soft, washable lining on the interior.

## 2 Intended use

### 2.1 Indications for use

The product is to be used **exclusively** for orthotic treatment of the head.  
The orthosis must be used in accordance with the indications.

### 2.2 Indications

- Use for children aged 3 to 18 months
- Nonsynostotic deformities of the skull: Plagiocephaly, symmetrical or asymmetrical brachycephaly
- Therapy after surgical operation of craniosynostosis with residual deformity of the skull

### 2.3 Contraindications

Not for use on infants with pre-surgical craniosynostosis or hydrocephalus.

## 3 Safety

### 3.1 General safety instructions

#### WARNING!

---

#### **Risk of severe injuries**

- ▶ The product may be used only by the person it was made for.
- ▶ Observe the notices regarding intended use and the use of the product.
- ▶ Do not use the product as a protective helmet (e.g. for cycling). The product has no protective effect.
- ▶ Check the product for damage prior to each application.
- ▶ Evaluate the structural integrity and fit of the orthosis to reduce the potential for the orthosis to slip out of place which could cause asphyxiation or trauma to the infant's eyes or skin.
- ▶ Do not continue to use a damaged product, and contact your orthotics and prosthetics professional (O&P professional).
- ▶ The expandable clasp is designed to accommodate the infant's growth; if the clasp reaches maximum expansion during treatment, contact your O&P professional for a follow up appointment.
- ▶ Evaluate the infant's skin at frequent intervals for signs of irritation or breakdown.
- ▶ Check the areas of skin that come into contact with the product before and after each use.
- ▶ Only use the product on intact skin.
- ▶ Note that using the product on a shaved head may cause ingrown hairs.

#### **Potential adverse effects**

- ▶ Skin irritation or breakdown.
- ▶ Potential adverse effects caused by extending treatment against medical advice. Extended treatment can cause skin irritation, skin breakdown and may result in excessive pressure.

#### NOTICE!

---

#### **Risk of product damage and limited functionality**

- ▶ During normal use, the product should not be exposed to temperatures lower than -10 °C (14 °F) and higher than +45 °C (113 °F).
- ▶ Clean the product if it was in contact with salt water or water containing chlorine.
- ▶ Observe the instructions for cleaning and care.

## 4 Scope of delivery

| Quantity | Designation                                 | Reference number |
|----------|---|------------------|
| 1        | Instructions for use (user)                 | -                |
| 1        | MyCRO Band with lining                      | -                |
| 1        | Spare lining (additional lining if ordered) | -                |
| 5        | Spare elastic closure band                  | 24Z2             |

## 5 Handling

### **⚠ WARNING**

#### **Failure to comply with the examination intervals**

Risk of injury due to improper use of the product

- ▶ Comply with the examination intervals established with your O&P professional in order to ensure the safe use of the product.

### **INFORMATION**

Wearing the orthosis in a child car seat is permissible. The child car seat must be adjusted so the child can sit in it properly with the orthosis.

During the first few days, the orthosis is worn for shorter periods of time so your child can become accustomed to it (see the table below). The orthosis should subsequently be worn all day and all night (23 hours minimum) in order to obtain the best possible results. The orthosis should be worn with as few interruptions as possible, e.g. removed only for bathing or treatment by a physiotherapist. Wearing the orthosis can cause increased perspiration during the first two to three weeks. This is normal and decreases over time.

If treatment is required for other illnesses such as torticollis during therapy with the MyCRO Band, the orthosis may be removed briefly.

If wearing the orthosis is interrupted for an extended period of time, another familiarisation period may be required.

**CAUTION: If there is a problem with the orthosis make an appointment with your O&P professional immediately . Do not wait until your next scheduled appointment. It is important that the orthosis is worn consistently.**

| Day of use | Daily duration of use             |
|------------|-----------------------------------|
| 1          | 2 hours                           |
| 2          | 4 hours                           |
| 3          | 6 hours (including naps)          |
| 4          | 8 hours (including naps)          |
| 5+         | Day and night (at least 23 hours) |

## 5.1 Application

### **⚠ WARNING**

#### **Fastening strap not glued to the orthosis**

Danger of suffocation or swallowing if the fastening strap gets into the mouth

- ▶ Do not use the orthosis, and contact your O&P professional to have the fastening strap glued.

### **⚠ WARNING**

#### **Wearing the orthosis in case of fever**

Risk of injury due to overheating

- ▶ Stop using the orthosis in case of fever.

### **⚠ WARNING**

#### **Incorrect or excessively tight application of the product to the body**

Risk of pressure points and constriction of blood vessels and nerves due to improper application

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ Do not continue to use the product if it fits incorrectly.

- 1) Check the following points before putting on the orthosis:
  - The baby is in a stable position with no interfering objects or items near the head.
  - Any lotions or creams used have been absorbed by the skin.
  - The orthosis is in proper condition with no cracks or other damage.
  - The lining fits inside the orthosis without wrinkles and projects slightly beyond the hard edge all around.

- 2) Open the elastic band of the closure (see fig. 1).

- 3) Carefully pull the orthosis apart at the closure (see fig. 2).

- 4) Align the orthosis and place it onto the baby's head (see fig. 3).  
 5) Check the fit of the orthosis:  
 → The arrowhead on the front should form a vertical line with the middle of the nose.  
 → The orthosis has to leave the ears and eyes uncovered.  
 → The lining should project beyond the edge of the orthosis all around and protect the skin against contact with the plastic.

- 6) **If the orthosis fits correctly:** Hold the orthosis carefully so it cannot shift and fasten the closure. A gap in the closure is normal and no cause for concern.  
 7) Face the flap of the elastic band up and verify that the elastic band fits properly in the guide of the closure.  
 8) **CAUTION! To avoid injuries, always open the closure before positioning the orthosis.**  
**If the orthosis does not fit correctly or has shifted:** Repeat the previous steps.

## 5.2 Removal

### INFORMATION

Redness of the skin after removing the orthosis is not unusual, and should go away within 2 hours. If the redness does not go away, or if pressure points develop, contact your O&P professional.

- > **Prerequisite:** The child is in a stable position with no interfering objects or items near the head.
- 1) Open the elastic band of the closure.
- 2) Carefully pull the orthosis apart at the closure.
- 3) Take the orthosis off the child's head.  
**TIP:** Gently massage the child's head with your fingertips after removing the orthosis in order to promote self-perception.
- 4) Remove the padding from the orthosis and clean it (see page 15). Insert the replacement padding into the orthosis.
- 5) Fasten the orthosis closure.

## 5.3 Changing the lining

The lining is attached to the orthosis with micro-hook. It has to fit inside the orthosis without wrinkles and must be in contact with the orthosis over the entire surface area.

### Removing the lining

- 1) Detach the folded lining in the neck area.
- 2) Starting at the closure, detach the lining all around the orthosis.

### Inserting the lining

- 1) Roll up the lining into a tight roll (see fig. 5).

- 2) Position the lining at the closure of the orthosis. Carefully unroll the lining, pressing it into the orthosis without wrinkles (see fig. 6).
- 3) Ensure that the lining projects slightly beyond the hard edge all around.

- 4) In the neck area, fold over the lining to the outside and press it into place (see fig. 7).

## 6 Cleaning

### NOTICE

#### Improper cleaning

Damage to the product due to improper cleaning

- ▶ Only clean the product with the approved cleaning agents.
- ▶ Only clean the product by hand.

### INFORMATION

Remove the lining from the orthosis before cleaning. Clean the lining and the shell of the orthosis separately.

> **Recommended cleaning agent:** pH neutral soap

- 1) Clean the product by hand with clean water and a pH neutral soap.
- 2) Rinse the soap away with clean water.

- 3) Dry the product with a soft cloth.
- 4) Allow to air dry fully before donning.

## 7 Disposal

Return the product to the O&P professional for disposal.

## 8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

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